

# EZ-Fit VT™ Compressive Calf Sleeve

## Compatibility:

This product is designed for use with the Triple Play-VT Vascular Therapy System

## LOT:

**DESCRIPTION:** Segmental pneumatic appliance used with pneumatic compressor, half leg. The wrap/sleeve has three (3) chambers that fill from the distal upward toward the proximal end of the extremity.

## INDICATIONS FOR USE:


- Aid in the prevention of DVT
- Enhance Blood Circulation
- Diminish post-operative pain and swelling
- Reduce wound healing time
- Aid in the treatment and healing of: stasis dermatitis, venous stasis ulcers, arterial and diabetic leg ulcers, chronic venous insufficiency and reduction of edema in lower limbs

## CONTRAINDICATIONS:

The TriplePlay VT must not be used to treat the following conditions:

- Persons with suspected, active or untreated: deep vein thrombosis, ischemic vascular disease, severe arteriosclerosis, pulmonary edema, congestive heart failure, thrombophlebitis or an active infection;
- On a leg where sleeves would interfere with the following conditions: vein ligation, gangrene, dermatitis, open wounds, a recent skin graft, massive edema or extreme deformity of the leg;
- On patients with neuropathy;
- On extremities that are insensitive to pain;
- Where increased venous or lymphatic return is undesirable

## ATTACHING THE LEG SLEEVES:

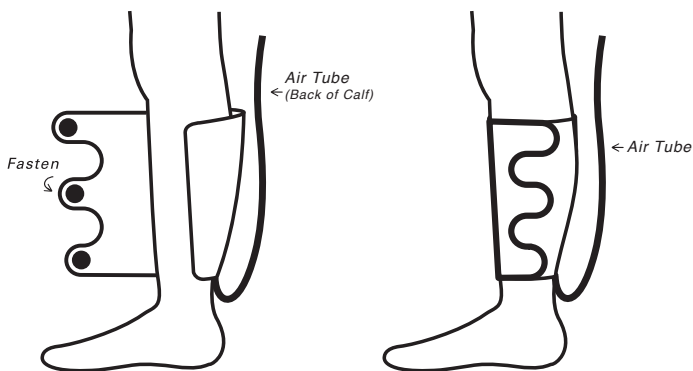

1. Make sure that the securing tabs are on the inside of the sleeve relative to your leg.
2. Place the sleeve so the middle of the bladder outline is against the back of your calf with the air tube facing downward.
3. Wrap the sleeve around your calf **so that it is a snug fit** and fasten the securing tabs.
4. Connect the hose to the Triple Play VT® unit.

**WARNING:** If pulsations or throbbing occur, the sleeve is wrapped too tightly. Remove the sleeve and re-wrap for comfort. If pain or swelling occurs, discontinue use and notify your medical professional immediately.